## ESPAÑOL

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## ENGLISH

NOBIL GRIP is a medical device pursuant to Regulation (EU) 2017/745.

**Intended use:** Suitable for bonding of threaded parts of attachments and anchors, prosthetic screws and orthodontics expansion screws. NOBIL GRIP assures the screwing in mouth over time, however maintaining the possibility of unscrewing and interchangeability.

**Users:** Medical device for professional dental technician and dental use only.

**Patient group:** Adult patients with problems to the masticatory system, who require clinical dental treatment to restore functions through the application of dentures.

**Clinical benefit expected:** Recover the functionality and aesthetics of the masticatory system.

**Contraindications:** Do not use the device outside of its intended use.

**Warnings:** Check the integrity of the package before use; do not use the product if you notice any detached parts, fractures or breakages.

Perform the bonding operations always outside of the oral cavity and wearing suitable personal protective equipment. Properly remove excess material by flushing under running water. To prevent material from obstructing the nozzle, prevent it from coming into contact with metal surfaces during dispensing. Do not use the product on plastic materials.

**Storage:** Store the product in its original container, well sealed, at a temperature ranging from 8 °C to 21 °C. Storing the product at temperatures lower than 8 °C or higher than 21 °C can affect its performance.

**Waste disposal:** The device must be disposed of in compliance with current regulations. Please consult the safety data sheet for further information.

**Disclaimer:** NOBIL GRIP manufactured by Nobil Metal must be used by specialised personnel aware of the technical protocols of use and capable of recognising any device failures. Nobil Metal will not be held liable for direct and/or indirect damage resulting from malpractice of the user and/or improper use of the device.

**Notice relating to serious incidents:** Whenever, during use of the device or in relation to it, a serious incident should occur, the patient and/or user must report it and to the Manufacturer Nobil Metal SpA and to the competent Authorities of the Member State where the event occurred, specifying the code and batch of the implicated product.

**Summary of safety and clinical performance:** The summary of the safety and clinical performance (SSCP) for NOBIL GRIP manufactured

by Nobil Metal can be consulted on the European medical devices database (Eudamed) at the following link: [ec.europa.eu/eudamed](http://ec.europa.eu/eudamed). The search can be made by means of the basic UDI-DI included in the EU Declaration of Conformity.

### PROCESS INSTRUCTIONS

NOBIL GRIP is an anaerobic curing glue.

The product does not change in the mouth and is non-toxic.

**PREPARATION OF THE BONDING SURFACES:** Clean and dry perfectly the parts to be bonded.

**APPLICATION:** Apply a small amount of NOBIL GRIP at 360° on the threads of the tap, leaving the first thread free. Apply enough product to fill the thread.

Screw the parts and immediately remove the excess material.

The prosthesis can be applied immediately after. The maximum setting is achieved after about three hours.

The cured product can be mechanically removed with a wire brush.

### SAFETY INFORMATION

#### Warning

H319 Causes serious eye irritation.

H317 May cause an allergic skin reaction.

H335 May cause respiratory irritation.

H412 Harmful to aquatic life with long lasting effects.

P261 Avoid breathing vapours.

P280 Wear protective gloves.

P273 Avoid release to the environment.

P333+P313 If skin irritation or rash occurs: Get medical advice/attention.

P337+P313 If eye irritation persists: Get medical advice/attention.

P501 Dispose of residues in compliance with current standards.

Contains: Hydroxypropyl methacrylate, 2,2-Ethylendiolodietil dimethacrylate, Methacryloyloxyethyl succinate, cumene hydroperoxide, 2-Hydroxyethyl methacrylate, 1-Acetyl-2-Phenylidrazina, Propylene glycol dimethacrylate

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