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MUA (Multi Unit Abutment)

MUA (Multi Unit Abutment) Uniabutment 20°

MUA (Multi Unit Abutment) 4.1 Slim Fit

#### DEVICE DESCRIPTION AND VARIANTS:

MUA (Multi Unit Abutments) are Medical Devices (MDs) designed to connect the dental implant to the prosthetic component in screw-retained restorations.

They are available in several configurations based on: compatibility with original implants, height, angulation, type of connection (monoblock and/or two-piece), and in the following versions:

- MUA (Multi Unit Abutment)
- MUA (Multi Unit Abutment) Uniabutment
- MUA (Multi Unit Abutment) Slim Fit

Each MD is supplied with its corresponding prosthetic screw.

In the straight configuration, the MD features an external thread that directly engages with the implant; the hexagon on the upper part of the cone allows tightening using a driver.

In the angled configuration, the MD has a non-rotational connection at the base that mates with the implant and is secured using a dedicated primary screw. Please refer to website

<https://naspeedyshop.it/index.php?lang=en> to verify the specifications of each code.

#### INTENDED USE AND EXPECTED CLINICAL BENEFIT:

These MDs are intended for use in prosthetic rehabilitation of partially or fully edentulous patients, with the aim of restoring masticatory function, phonetics, and aesthetics.

They allow for correction of implant angulation discrepancies, promote passive fit of the prosthetic structure, and enhance tissue integration by shifting the prosthetic interface coronally.

#### INDICATIONS FOR USE:

The MDs are non-sterile and single-use, preformed components intended to support screw-retained dental prostheses (crowns, bridges) on implants.

They are applied via screw-retention, enabling integration between the implant and the prosthetic component.

#### CONTRAINDICATIONS:

- Do not use on patients allergic or intolerant to the materials composing the devices.
- Verify compatibility with any other existing prosthetic materials already present in the oral cavity.

**BASIC UDI-DI:** 8053876-01-004N7

#### TARGET USERS:

Use is restricted to Dental Technicians and Dentists who are properly trained and licensed to practice in appropriate clinical settings.

#### TARGET PATIENT:

Patients with one or more dental implants.

#### MATERIAL:

MUA/Screw: Ti6Al4V-ELI (Titanium alloy)

#### PROCEDURE FOR USE:

##### In the Dental Laboratory:

1. Screw the MD onto the physical model from the analog or digital workflow.
2. After placement, check for any interference between the MD and the model. If interference is present:
  - Remove excess material causing the interference
  - Repeat placement
3. Screw the MD onto the analog using the supplied screw.
4. Proceed with fabrication of the prosthetic restoration (support structure and aesthetic components).
5. Verify marginal seal and passive fit of the structure/restoration on the MD.

**WARNING:** The only thermal treatment allowed for the MD is sterilization.

##### In the Dental Clinic:

1. Sterilize the MD before intraoral placement (see "Cleaning, Washing, and Sterilization").
2. Check proper seating of the MD on the implant. If the MD is two-piece, ensure all parts are properly seated together and on the implant.
3. Screw the MD(s) onto the implant(s), adhering to the torque specified on the label to avoid damage to the prosthetic system.
4. Verify marginal seal and passive fit of the structure/restoration on the MD. Ensure compatibility of all prosthetic materials used in the restoration.
5. Before final placement of the structure/restoration, seal the screw channel.

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**WARNING:**

It is the clinician's responsibility to:

- Determine the appropriate screw channel sealing material and clinical procedure.
- Instruct the patient on proper hygiene maintenance.

**WORKABILITY LIMITATION:**

The MD is non-modifiable.

**CLEANING, WASHING, AND STERILIZATION:**

The MDs are supplied cleaned and washed according to New Ancorvis' validated protocol.

They are non-sterile; proper cleaning, disinfection, and sterilization before clinical use are essential to reduce patient risk and to prevent infections or local inflammation that could compromise the biological integrity of peri-implant tissues.

**In the dental laboratory:**

During laboratory procedures, MDs may be disinfected using cold disinfection methods.

The prosthesis containing New Ancorvis MDs must be disinfected before delivery to the dental clinic.

It is the Dental Technician's responsibility to identify a disinfection method compatible with all prosthetic materials used.

**In the Dental Clinic:**

Prior to patient use, the prosthesis containing the MD(s) must be sterilized according to the instructions provided by the laboratory.

**INTRAORAL USE OF A SINGLE DEVICE:**

In case of intraoral use of a single MD (e.g., during try-ins), follow the procedure below:

- The MDs are supplied non-sterile; they must be packaged and sterilized before use on the patient. Before sterilization, remove the MD from its original packaging and place it in sterilization pouches compatible with the described steam sterilization method.
- Pouches must be sealed using a validated method by the user.
- Each pouch must be placed in a **saturated** steam autoclave and subjected to a 134°C cycle for at least 5 minutes.
- After sterilization, store the pouch in a cool, dry place away from heat, light, and dust. Open the pouch only at the time of use.

**COMPATIBILITY: Implants, prosthetic components, and instrumentation**

Compatibility with the original implant (Brand, Model, Platform) is indicated on the MD label.

It is the responsibility of the user (Dentist or Dental Technician, as applicable) to select the appropriate New Ancorvis prosthetic component compatible with implants, prosthetic components, and instruments from other manufacturers.

Prior to use, verify the information regarding device compatibility, masticatory load, and restoration design to determine appropriate component.

**Prosthetic Instrumentation:**

MDs must be used in combination prosthetic instruments (positioners, direction indicators, screwdrivers, adapters, torque wrenches) that are suitable for and compatible with the screw used with the MD.

New Ancorvis provides instruments compatible with its MDs. Refer to <https://naspeedyshop.it/index.php?lang=en> for compatibility specifications.

**WARNING:** All prosthetic instruments used in the clinic must be sterilized before use, according to the manufacturer's instructions. Do not use damaged instruments or those incompatible with the application procedure.

**CAUTIONS:**

- Screws, screw seats, and connections must NEVER be modified in any way.
- Inadequate tightening torque may compromise the mechanical integrity of the components, connection stability, and sealing.

**PRECAUTIONS:**

- Do not use MDs outside their intended use.
- Before each use, check the integrity of the MDs; do not use if any anomalies are detected.
- Due to their small size, take care during clinical procedures to prevent accidental ingestion by the patient.

**REUSE AND RELATED PRECAUTIONS:**

These devices are single-use only; any reuse exposes the patient to the risk of cross-contamination.

Therefore, the appropriate symbol is displayed on both the label and the packaging.

**STORAGE:**

Store in the original packaging until use, in a cool, dry place, protected from moisture, dust, or other contaminants.

**DISPOSAL:**

Dispose of MDs in accordance with applicable regulations. If used, they must be discarded following current regulations for biologically contaminated waste.

**WARRANTY:**

New Ancorvis provides a lifetime warranty on its compatible prosthetic components.

**DISCLAIMER OF LIABILITY:**

The Gemini Line components manufactured by New Ancorvis must be used by qualified professionals familiar with implant and prosthetic restoration protocols and capable of identifying any device defects. New Ancorvis disclaims any liability for direct and/or indirect damages resulting from user negligence, unauthorized modifications to the original device shape beyond indicated limits, improper use, or incorrect storage and handling.

**NOTICE REGARDING SERIOUS INCIDENTS:**

REV. 23/09/2025

In a serious incident occurs during or in relation to the use of the device, the patient and/or user must report it to the competent authority in the country where the event occurred and to the manufacturer, NEW ANCORVIS srl, at the following address: [qualita@newancorvis.eu](mailto:qualita@newancorvis.eu), specifying the product code and lot number involved.

#### SSCP:

The SSCP (Summary of Safety and Clinical Performance) document is available on EUDAMED (<https://ec.europa.eu/tools/eudamed>). Until the platform is fully operational, it can be requested via email at [qualita@newancorvis.eu](mailto:qualita@newancorvis.eu).

#### MRI SAFETY INFORMATION (For implantable devices).

The MDs have not been evaluated for safety and compatibility in a magnetic resonance (MR) environment. They have not been tested for heating, migration, or image artifacts in MR settings. Scanning a patient with an MD may cause injury. **Important:** Both the patient and the MR operator must be informed of this risk.

#### SYMBOLS:

For an explanation of the symbols, please refer to the document 'Legenda simbologia New Ancorvis'.

MUA (Multi Unit Abutment)  
MUA (Multi Unit Abutment) Uniabutment 20°  
MUA (Multi Unit Abutment) 4.1 Slim Fit

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