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TECHNICAL DATA SHEET

Commercial name	Calibrated premilled Code: PRCA
Intended use	Device intended for the calibration of premilled holders mounted in milling centers.
Description of the device and how to use it	The PRCA device is made of brass. The upper part has a cylindrical shape with a diameter of 12 mm and a length of 20 mm. It is designed to detect and correct the actual positions of the Premilled relative to their theoretical positions, which may be inaccurate due to misalignment of the milling machine, such measurements must be carried out using a digital micrometer. The calibration function is consistent with the use of devices and procedures aimed at detecting and correcting deviations from a nominal reference. The device is used during milling centre verification and calibration operations; it is positioned within the machine's control system to detect any differences between the theoretical and actual positions of the Premilled. Based on the deviation detected, the operator corrects the machine's centring or offset, thereby realigning the actual positions with the nominal ones.
Contact area features and application time	The device does not come into contact with the patient, but only with the milling machine.
Technical features	
Material	Brass
Screw tightening	Not Applicable
Single use	YES
Status	Non-sterile